



# EndoRing® FileCaddy®

## Endodontic File Storage Device



The EndoRing FileCaddy organizer provides a unique bulk storage solution for all endodontic files. The clinician places multiple files securely into the special foam insert, closes the lid and steam autoclaves the entire FileCaddy organizer—no bag or wrap required. Single handed operation allows easy opening of the protective lid to access the processed files at chair side. Files are easily identified, selected and transferred to the companion Jordco EndoRing II organizer for the current endodontic procedure. The FileCaddy organizer disassembles easily to allow replacement of the foam insert.

### TECHNICAL SUMMARY

**FileCaddy Case:** The EndoRing FileCaddy case is constructed of a special glass reinforced thermoplastic resin that can be safely steam autoclaved processed, do not exceed 275°F (135°C). There are three molded components that make up the FileCaddy case; a molded base, retainer and lid. All the components are engineered to easily and reliably assemble. A specially designed retainer locking mechanism securely holds the foam insert onto the base. The color of the case is a light beige tone. Size (assembled): 4 5/8" w x 4 5/8" d x 3" h.

**FileCaddy Foam Insert:** The disposable EFC Foam Insert is constructed of Jordco's proprietary open cell foam that is also used in our Jordco® e-Foam® Inserts. The EFC Foam Insert and case can be steam autoclaved processed, do not exceed 275°F (135°C). The foam inserts are available in blue or yellow color.

\* The Foam Insert should be replaced every two autoclaving cycles to prevent discoloration of the EFC resin case. If discoloration occurs, simply disassemble the EFC and soak the components in household bleach over night to remove discoloration.

Discover our complete line of quality Jordco products at [www.jordco.com](http://www.jordco.com) or contact your dental supply dealer



Jordco, Incorporated  
595 NW 167th Ave., Beaverton, Oregon 97006 USA  
(800) 752-2812 [www.jordco.com](http://www.jordco.com)  
Visit our website for patent and trademark information

### PRODUCT SPECS

**Name:** EndoRing FileCaddy  
**REF:** EFC-s (Kit)  
EFCF-s (Refill Foam Combo 6 Blue / 6 Yellow)  
EFCFB-s (Refill Foam 12 Blue)  
EFCFY-s (Refill Foam 12 Yellow)  
**Weight:** Kit: 1 lb 5 oz  
Refill Packs: 2.6 oz

### PACKAGE CONTENTS

- Kit**
- 1 EndoRing FileCaddy Assembly
  - 12 EndoRing FileCaddy Foam Inserts (Combo Pack)
- \* Files and EMT-s not included

### Refill Packs

- 12 EndoRing FileCaddy Foam Inserts

### FEATURES AND BENEFITS

#### Locking Lid

Files are held securely in place between procedures and will not spill out if the FileCaddy organizer is dropped.

#### Autoclavable and Durable

The FileCaddy organizer is constructed of a medical grade resin that can withstand repeated steam autoclaving. The resin is glass reinforced for added durability and longevity.

#### 4 Bulk Storage Compartments

The FileCaddy organizer is divided into 4 convenient bulk storage compartments to help make file organization easy. The separate compartments simplify organization and retrieval of instruments. Over 100 instruments can be safely stored in the FileCaddy organizer for steam autoclaving.

#### Disposable Foam Inserts

The foam inserts are disposable to reduce cross contamination\*.

#### Two Colors of Foam Inserts

Two colors are helpful to organize/color code file setups.

#### Secure File Loading

Foam inserts eliminates the risk of spilling and losing files.

#### One-Hand Operation

One hand operation allows for easy opening of the locking lid to access files and contents.

#### Bulk File Storage

Ideally store new or unused files securely in the FileCaddy. During the procedure files are easily retrieved for use and can be used as a companion to the EndoRing.