



Instructions For Use

e-Ruler®

The e-Ruler dental device is used by to measure files and instruments used during endodontic procedures. This compact ruler is independently worn on the finger of dental professional while performing an endodontic procedure.

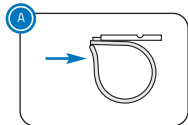

A. Before assembly, choose one of three available ring sizes (small, medium or large) that best fits your index finger. Slide the selected ring into the ruler channel located underneath. To secure the locking mechanism, press the ring and ruler against a flat surface. The ring will be permanently attached to ruler. **Do NOT attempt to remove.**

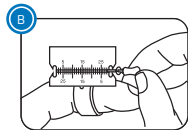

B. When using rubber stops, utilize the stop locks on the ruler (only on Stop-Lock version) to increase or decrease the instrument's working length. The ruler performs well with paper points, gutta percha and many other endodontic instruments.

△ PROCESSING INSTRUCTIONS

Precleaning Instructions: Rinse the e-Ruler under tap water to remove gross and visible soils. Use a scrub brush if necessary to remove visible soils. Place the e-Ruler in an ultrasonic bath with **only an enzymatic detergent** prepared to the manufactures directions. Fully immerse the item and soak for 1 minute. Then sonicate the e-Ruler for 5 minutes and remove and rinse for 15 seconds with tap water. Inspect for any soils and repeat if any soils are visible.

Steam Autoclave Processing Instructions: Place the e-Ruler in an FDA approved or ISO 11607 compliant autoclave pouch/bag. During steam autoclave processing, place all components near center of the chamber. Do not exceed temperature of 275 °F (135 °C). (1805)

	Gravity Displacement Autoclave	Pre-Vacuum Autoclave
Temperature	121 °C (250 °F)	132 °C (270 °F)
Process Time	30 minutes	4 minutes
Drying Time	30 minutes	20 minutes

△ **Warning:** After multiple uses, if any component of the e-Ruler demonstrates cracking, fractures or any loss of integrity dispose and replace with a new e-Ruler.

△ **Warning:** Do not place any components into Surgical Milk, "cold sterilizing solutions" or instrument dips prior to autoclave processing. Only use **enzymatic ultrasonic detergents** when precleaning the EndoRing cup and the metal ruler in ultrasonic cleaners. Report any serious incident with this device to Jordco Corporation and the governing medical device authorities in your country. (2006)

Symbols Glossary Available at: www.jordco.com/symbols

Jordco products are covered by one or more U.S. or foreign patents or patent applications owned by Jordco, Inc. of Beaverton, OR, USA. See www.jordco.com for details. JORDCO, ENDORING, E-FOAM, ENDOGEL, E-DX, and FILECADDY are trademarks of Jordco, Inc., registered in the United States and/or other countries. The blue color of the e-Foam inserts and the endodontic organizer, the yellow color of the e-Foam inserts, and the colors yellow and blue-yellow together on e-Foam insert packaging, are all trademarks of Jordco, Inc., registered in the United States. See the above website for further information. (1704)

REV2009

e-Ruler™ both Stop Lock & No-Stop Lock versions

Please read before using the e-Ruler.



Jordco, Incorporated

Beaverton, Oregon USA 97006

503/531-3904 TEL, 503/531-3757 FAX, 800/752-2812 toll free USA

www.jordco.com



1 Before assembly, choose one of three available ring sizes (small, medium and large) that best fits your index finger.



2 Slide the selected ring into the ruler channel located underneath. To secure locking mechanism, press ring and ruler against flat surface. Ring will be permanently attached to ruler—do NOT attempt to remove.



1 When using rubber stops, utilize the stop locks on the ruler (only on Stop Lock version) to increase or decrease the instrument's working length. The ruler performs well with paper points, gutta percha and many other endodontic instruments.